

# Disposable Dressing Set

## 一： Product name, model and specification

Product name: disposable dressing Set

Model specifications: Cleaning type, Debridement type, PICC(central venous catheter) type

## 二： Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

1. Name: Hunan Zhenheyikang Medical Instrument Co., LTD

2: Address: NO.6 Building Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan

3:Production address:NO.6 Building, NO.8 Building,3rd and 4th floor Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan,NO 19 Chutian Road, Yuetang District, Xiangtan City, Hunan

4:Contact number:+86 17711627084

5:After-sales service unit:Hunan Zhenheyikang Medical Instrument Co., LTD

三、 Manufacturing License NO : Hunan Medical Products Administration Production Licence No.20150013

四、 Product Registration Certificate No、 Product Technical Requirements No: HNFDA Certified 20182140083

## 五、 Structure and composition, scope of application

Cleaning type: tray, plastic tweezers, medical gauze block, iodophor cotton ball, alcohol cotton ball, towel, disinfection forceps; Debridement type: surgical blade, medical suture needle, non-absorbable surgical suture, tray, plastic tweezers, medical gauze block, iodine cotton ball, alcohol cotton ball, towel, disinfection forceps; PICC(central venous catheter) type: blue wrapping paper, disinfection cotton swab, disposable medical alcohol disinfection tablet, disposable sterile rubber surgical gloves, pull adhesive tape, self-adhesive transparent dressing, non-woven gasket composition. The accessories in the package should be pollution-free, no damage, no odor, tasteless phenomenon. Plastic accessories should be smooth, uniform color, without burrs, flash and cracks. Product aseptic supply, ethylene oxide sterilization, ethylene oxide residue should be  $\leq 10\mu\text{g/g}$ .

For clinical patients in medical institutions to use for dressing change surgery, iodophor cotton ball, alcohol cotton ball for hand, skin and mucous membrane disinfection.

六： Use method: open the bag can be used.

## 七： Contraindications, precautions, warnings and reminders

Product contraindications: None

Damaged packaging prohibited use, expired products prohibited use;

Disposable use, please destroy after use;

Avoid high temperature, humidity, direct sunlight;

#### **八： Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels**




一次性使用




参考使用说明书



包装破损切勿使用

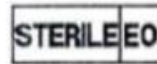

环氧乙烷灭菌

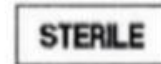

无菌

#### **九. Installation and use instructions or illustrations**

This product does not need to be installed, open the bag can be used.

#### **十, Product maintenance and maintenance methods, special storage, transportation conditions, methods**

Products should be stored in relative humidity  $\leq 80\%$ , non-corrosive gas and well-ventilated indoor, ordinary transport, in the transportation, storage process should be kept clean and pollution-free, strictly prohibited rain and snow immersion.

#### **十一： Date of production, shelf life or expiration date**

Valid for two years, production batch number, production date, expiry date please refer to the small package or qualification label.

#### **十二, Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description**

This product has no accessories

#### **十三. Date and version of preparation or revision of the specification**

This specification was revised in February 2023, version 2023.