

Isolation shoe cover

一、 Product name, model and specification

Product name : Isolation shoe cover

Model : ordinary PE type、 ordinary PP type

Specification : S、 M、 L

二、 Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

1、 Name: Hunan Zhenheyikang Medical Instrument Co.,LTD

2、 Address: No.6 Building Jingxiang Energy, No.55 Xianguang east road, Gaoxin district Xiangtan, Hunan

3、 Production address:NO.6 Building, NO.8 Building,5th floor Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan

4、 Contact number : +86 17711627084

5、 After-sales service unit: Hunan Zhenheyikang Medical Instrument Co.,LTD

三、 roduction Registration Certificate No: Xiangtan Medical Products Administration
Production Registration NO 20140004

四、 Product Registration Certificate No、 **Product Technical Requirements** No:
XTFDA Certified 20200044

五、 Product description, intended use

The main materials of the product are plastic film and PP material, made by heating and sewing; with sufficient mild and barrier performance, disposable use, non-sterile supply.

Medical personnel to prevent contact with patients with potential infectious blood, body fluids, secretions, etc., play a barrier and protective role.

六、 Use method : Open the bag can be used, directly to treat the wound

七、 Contraindications, precautions, warnings and reminders

Product contraindications: None

Avoid high temperature, humidity, direct sunlight

Damaged packaging prohibited use, expired products prohibited use

Disposable use, please destroy after use

It is forbidden to approach high temperature heating, corrosive objects, do not approach open fire.

八、 Interpretation of graphics, symbols, abbreviations and other contents used medical device labels



一次性使用



参考使用说明书



包装破损切勿使用

九、 Installation and use instructions or illustrations

Choose the right specifications to wear it directly. It is recommended to use no more than 8 hours.

十、 Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in a non-corrosive gas and well-ventilated room, transported by ordinary transportation means, keep clean and pollution-free, rain and snow immersion.

十一、 Date of production, shelf life or expiration date

Valid for three years, production batch number, production date, expiry date please refer to the small package or qualification label.

十二、 Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description

This product has no accessories

十三、 Date and version of preparation or revision of the specification

This specification was revised in May 2023, version 2023.