

Medical Tape

一、 Product name, model and specification

Product name : Medical Tape

Model : Common type, desensitization type, breathable type

Specification : 1~50cmX100~1000cm

二、 Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

1、 Name: Hunan Zhenheyikang Medical Instrument Co.,LTD

2、 Address: No.6 Building Jingxiang Energy, No.55 Xianguang east road, Gaoxin district Xiangtan, Hunan

3、 Production address:NO.6 Building, NO.8 Building,5th floor Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan

4、 Contact number : +86 17711627084

5、 After-sales service unit: Hunan Zhenheyikang Medical Instrument Co.,LTD

三、 Production Registration Certificate No: Xiangtan Medical Products Administration
Production Registration NO 20140004

四、 Product Registration Certificate No、 **Product Technical Requirements** No:
XTFDA Certified 20190014

五、 Product description, intended use

The product is made of polyethylene PE with the base material and medical adhesive. No direct contact with the wound surface, and the paste part is intact skin. Non-sterile provided, single use.

Used to attach the dressing to the wound surface or to fix other medical devices to a specific part of the human body.

六、 Use method : Open the bag can be used, directly to treat the wound

七、 Contraindications, precautions, warnings and reminders

Product contraindications: people with skin allergies use cautiously

It is strictly prohibited to use after mildew, damaged packaging and expiration.

It is a disposable product and cannot be reused.

The waste should be disposed of according to the requirements of the hospital or the environmental protection department after using.

Avoid high temperature, humidity, direct sunlight.

八、 Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels



一次性使用



参考使用说明书



包装破损切勿使用

九、 Installation and use instructions or illustrations

This product does not need to be installed, open the bag can be used.

十、 Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in a non-corrosive gas and well-ventilated room, transported by ordinary transportation means, keep clean and pollution-free, rain and snow immersion.

十一、 Date of production, shelf life or expiration date

Valid for three years, production batch number, production date, expiry date please refer to the small package or qualification label.

十二、 Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description

This product has no accessories

十三、 Date and version of preparation or revision of the specification

This specification 参考 revised in May 2023, version 2023.