

Rubber Surgical Gloves

一、 Product name, model and specification

Product name: Rubber surgical gloves

Model: hemp surface, smooth finish, powder surface, Powder-free surface; sterilized, unsterilized;

Specification: 5、 5.5、 6、 6.5、 7、 7.5、 8、 8.5、 9、 9.5

二、 Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

1. Name: Hunan Zhenheyikang Medical Instrument Co., LTD

2. Address: NO.6 Building Jingxiang Energy, No.55 Xiaguang east road, Gaoxin district Xiangtan, Hunan

3. Production address: NO.6 Building, NO.8 Building, 3rd and 4th floor Jingxiang Energy, No.55 Xiaguang east road, Gaoxin district Xiangtan, Hunan, NO 19 Chutian Road, Yuetang District, Xiangtan City, Hunan

4. Contact number: +86 17711627084

5. After-sales service unit: Hunan Zhenheyikang Medical Instrument Co., LTD

三、 **Manufacturing License NO** : Hunan Medical Products Administration Production Licence No.20150013

四、 **Product Registration Certificate No、 Product Technical Requirements No:** HNFDA Certified 20202141644

五、 Structure and composition, scope of application

This product is made of natural rubber latex as raw material, according to the design is divided into straight gloves and fingers bent to the palm of gloves, according to the surface type is divided into hemp surface, smooth surface, powder surface and powder-free surface. Sterilized gloves are sterilized by ethylene oxide and the product is sterile.

The product is used for clinical surgery in medical units.

六、 **Use method:** open the bag can be used.

七、 Contraindications, precautions, warnings and reminders

- 1、 Allergy to natural rubber latex is prohibited;
- 2、 Should not be directly used in the operating room or invasive operation before sterilization;
- 3、 Marked with power products, The powder should be aseptically removed before surgery;
- 4、 Disposable use, please destroy after use;

5、The integrity of the packaging should be carefully checked before using, Prohibited to use if the initial packaging is damaged or there are foreign matter inside;

6、When using this product, choose the specifications suitable for your own, do not wear rings or other ornaments, and pay attention to trim nails, can not be cut by sharp objects;

7、The used products shall be placed in the special medical device recycling box designated by the medical department;

8、Avoid high temperature , humidity, direct sunlight;

八、 Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels



一次性使用

参考使用说明书

包装破损切勿使用

环氧乙烷灭菌

无菌

九、 Installation and use instructions or illustrations

This product does not need to be installed, open the bag can be used.

十、 Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in relative humidity $\leq 80\%$, non-corrosive gas and well-ventilated indoor, ordinary transport, in the transportation, storage process should be kept clean and pollution-free, strictly prohibited rain and snow immersion.

十一、 Date of production, shelf life or expiration date

Valid for two years, production batch number, production date, expiry date please refer to the small package or qualification label.

十二、 Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description

This product has no accessories

十三、 Date and version of preparation or revision of the specification

This specification was revised in February 2023, version 2023.